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Quick Start guide Product Features and Quick Usage Guide

1	Cartridge Insertion Slot	2	Plunger Position Indicator	3	Start Button
4	Charging Port (Type-C)	5	Battery Level Indicator (LED)	6	On/Off Button
7	Reverse Button				

copyright © 2024 PrimeDental all rights reserved.

Precautions Before Use

- 1. Check for any product damage, foreign substances, or impurities before using the product.
- 2. Do not reuse the mixing tip once it has been used.
- 3. Use the 50ml silicone cartridge device only for its intended mixing purpose.

Instructions and Operation Steps

- 1. Turn on the power switch
- 2. Before attaching the cartridge, ensure that the plunger is retracted.
- 3. Remove the cartridge cap and vertically insert the cartridge into the cartridge holder.
- 4. Attach the mixing tip to the cartridge.
- 5. Press the start button to operate the device.
- 6. When replacing the cartridge:
- 1 Press the reverse button to fully retract the plunger.
- ② Remove the cartridge by pulling it upwards.

Caution

- 1. This product is designed for C-type charging; please use the correct power cable during charging.
- 2. Keep the product away from flammable materials to prevent explosions or fires.
- 3. Protect the product from moisture to avoid electric shock or malfunction.
- 4. Handle the product carefully to avoid damage from drops.
- Do not place or use the product in high temperatures or direct sunlight to prevent internal malfunctions, smoke, or fire.
- Do not touch the ends of the charger with your fingers or conductive objects, as it may cause overheating or fire.
- 7. Do not frequently connect or disconnect the product or cable with wet hands to prevent the risk of electric shock,
- 8. If you are not using the product for an extended period, turn off the power switch and store it properly.
- 9. Do not disassemble the product. Unauthorized disassembly may cause malfunction, injury, electric shock, or fire, and it will void the warranty.
- Disconnect the charging cable when the battery is fully charged to avoid overcharging, which can shorten battery life.

Customer Service

If a malfunction occurs during product use, you are eligible for free service for 6 months from the date of purchase. However, if the malfunction is caused by external factors such as misuse or natural disasters, the repair will be charged even within the warranty period.